

HEINE BETA X Ophthalmoscope

THE MOST IMPORTANT FUNCTIONS AT A GLANCE

To ensure optimal battery performance and longevity,
fully charge your HEINE BETA X Ophthalmoscope
to 100 % before its first use.



QUALITY
MADE IN GERMANY



HEINE BETA X Ophthalmoscope

HEINE

focusCONTROL

Switch between examination of the posterior (F = fundus) and anterior segment (C = cornea) of the eye

Filter selection

-  red-free filter
-  without filter
-  polarisation filter



Aperture wheel

7 different apertures which can also be used with red-free filter or with polarisation filter



For quick and easy results in fundus examinations choose the settings marked with a white dot:

-  Filter: polarisation filter
-  Aperture: large spot
- F focusCONTROL: fundus



More information
on heine.com

